



EVER X-FLOW®

3D Printing Resin

Backed by a Strong In-House R&D Team

Advanced photopolymer formulations engineered for high-precision digital dental workflows.
Optimized for clinical and laboratory additive manufacturing.



Model Resin

This model resin delivers excellent printing accuracy and is suitable for producing dental restorative master models and detailed orthodontic models. With its optimal color and opacity, it produces models with well-defined contours and fine details, while providing excellent surface properties for scanning. It offers a reliable model base for the design and fabrication of a wide range of dental restorations and appliances.

Key Performance Benefits



Tear and wear resistance



Excellent pressure resistance



Fast curing



Low shrinkage



Good fault tolerance



Excellent detail reproduction



Specifications

Shades : Gray, Almond, Light Yellow

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	180-380 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	82-85 D (Shore D)	ASTM D2240
Tensile Strength	40-50 MPa	ASTM D638
Elongation at Break	8-13 %	ASTM D638
Flexural Modulus	2000-2400 MPa	ASTM D790
Flexural Strength	68-80 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Surgical Guide Resin

Surgical Guide Resin is a high-precision 3D printing material developed for the fabrication of implant surgical guides, gingivectomy guides, orthodontic bracket bonding guides, and bone reduction guides. It offers high transparency, mechanical strength, and dimensional stability. Printed parts feature a smooth surface, resist warping and maintain their accuracy after steam sterilization.

Key Performance Benefits



Resists warping and deformation



Good mechanical properties



High transparency for clear observation of oral structures



High stability and strength



Biocompatible



Specifications

Shade : Translucent white

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	300-450 mPa.s	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	80-82 D (Shore D)	ASTM D2240
Tensile Strength	40-50 MPa	ASTM D638
Elongation at Break	10-20 %	ASTM D638
Flexural Modulus	700-2000 MPa	ASTM D790
Flexural Strength	75-84 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Castable Wax Resin

This castable photopolymer resin is designed for dental investment casting applications, including crowns, removable partial denture frameworks, and other cast dental restorations. It delivers fine detail reproduction, a smooth surface finish, and excellent dimensional stability. Its clean burnout and low ash residue support accurate and consistent casting results.

Key Performance Benefits



Fine Detail
Reproduction



Excellent
Dimensional Stability



Clean Burnout with
Low Ash Residue



Specifications

Shades : Green

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	350 mPa.s	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	60-68 D (Shore D)	ASTM D2240
Tensile Strength	10-15 MPa	ASTM D638
Elongation at Break	10-20 %	ASTM D638
Flexural Modulus	450-680 MPa	ASTM D790
Flexural Strength	20-30 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Water-Washable Ortho Model Resin

Water-Washable Ortho Model Resin is a high-precision, rigid material developed for orthodontic diagnosis, treatment planning, and study model fabrication. It delivers fine detail reproduction, high surface hardness, and excellent dimensional and edge stability.



Key Performance Benefits



High-Precision
Detail Reproduction



High Surface
Hardness



Excellent
Dimensional Stability

Specifications

Shades : Apricot, Light Yellow, Gray

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	180-380 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	82-85 D (Shore D)	ASTM D2240
Tensile Strength	40-50 MPa	ASTM D638
Elongation at Break	8-13 %	ASTM D638
Flexural Modulus	2000-2400 MPa	ASTM D790
Flexural Strength	68-80 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Temporary Crown & Bridge Resin

Temporary Crown & Bridge Resin is designed for the fabrication of temporary crowns and bridges. It combines reliable mechanical strength, fracture resistance, wear resistance, and low water absorption. Available in multiple tooth shades, the material can be polished to achieve a smooth surface with natural tooth-like color and luster.



Key Performance Benefits



Reliable
Mechanical Strength



Good Wear
Resistance



Natural Tooth-Like
Aesthetics

Specifications

Shades : A1, A2, A3, BL1

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	350-400 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	80-90 D (Shore D)	ASTM D2240
Tensile Strength	65 MPa	ASTM D638
Elongation at Break	2.8-13 %	ASTM D638
Flexural Modulus	2000-3000 MPa	ASTM D790
Flexural Strength	80-120 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Permanent Crown & Bridge Resin

Our Permanent Crown & Bridge Resin combines high mechanical strength, biocompatibility, and precise marginal fit for durable restorations. Its wear resistance, color stability, and natural translucency deliver lasting aesthetics, while the smooth, easy-to-polish surface supports efficient digital dental workflows.

Key Performance Benefits



Long-Term Durability
High mechanical strength and wear resistance help withstand daily chewing forces for long-lasting performance.



Natural Aesthetics
Reliable color stability and natural translucency create a lifelike appearance.



Precise Marginal Fit
Precise margin and detail reproduction, with a smooth, easy-to-polish surface for improved fit and efficient production.



Specifications

Shades: A1, A2, A3, BL

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	800-3000 mPa.s	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	85-95 D (Shore D)	ASTM D2240
Tensile Strength	45-65 MPa	ASTM D638
Elongation at Break	2-5 %	ASTM D638
Flexural Modulus	6000-12000 MPa	ASTM D790
Flexural Strength	80-120 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Gingival Resin

Gingival Resin is designed for producing flexible and realistic gingival models. It combines natural gingival shades and textures with good elasticity and tear resistance, enabling accurate reproduction of soft-tissue details. The printed material is easy to handle and maintains reliable dimensional stability.

Key Performance Benefits



Superior elasticity



Excellent tear resistance



Low shrinkage, maintaining long-lasting flexibility



Good dimensional stability



Specifications

Shades : Dark Red, Pink, Light Pink

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	210-500 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	50-55 D (Shore D)	ASTM D2240
Tensile Strength	3.2-5 MPa	ASTM D638
Elongation at Break	5-15 %	ASTM D638
Flexural Modulus	500-880 MPa	ASTM D790
Flexural Strength	84 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Denture Base Resin

This 3D-printable denture base resin combines reliable mechanical performance with natural gingival aesthetics. Once cured, it presents a natural gingival shade and smooth surface while helping maintain color stability over time. Its low water absorption, fracture resistance, and dimensional stability support durable denture bases with an accurate and comfortable fit.

Key Performance Benefits



Natural Aesthetics
Reproduces natural gingival shades and textures for lifelike denture aesthetics.



Strength and Durability
Combines mechanical strength and fracture resistance to withstand daily functional loads.



Precise Fit
Low shrinkage and dimensional stability support an accurate and stable denture base fit.



Specifications

Shades : Dark Red, Pink, Light Pink

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	200-800 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	80-85 D (Shore D)	ASTM D2240
Tensile Strength	35-55 MPa	ASTM D638
Elongation at Break	5-12 %	ASTM D638
Flexural Modulus	1800-2500 MPa	ASTM D790
Flexural Strength	>65 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Soft Splint Resin

Soft Splint Resin is a flexible, transparent 3D printing material developed for fabricating comfortable occlusal splints and night guards. It balances flexibility and strength, with low shrinkage and excellent dimensional stability for a precise, comfortable fit.

Key Performance Benefits



High Transparency
Crystal-clear appearance for easy verification of fit and occlusion.



Flexibility & Comfort
Soft, resilient texture ensures comfortable long-term wear.



Wear Resistance
Durable against nightly occlusal loads for long-lasting use.



Specifications

Shade : Transparent

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	1000-8000 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	80-90 D (Shore D)	ASTM D2240
Tensile Strength	50-60 MPa	ASTM D638
Elongation at Break	2-5 %	ASTM D638
Flexural Modulus	2000-3500 MPa	ASTM D790
Flexural Strength	>80 MPa	ASTM D790

Flexible Denture Resin

Flexible Denture Resin is designed for removable partial and full dentures, offering natural gingival translucency and a soft, flexible texture. Its elasticity, tear resistance, and fracture resistance support comfortable wear and reliable long-term use. The material combines durability with natural aesthetics for flexible denture fabrication.

Key Performance Benefits



Natural Gingival Aesthetics
Natural gingival shades and translucency create a lifelike appearance.



Strength and Durability
Good tear and fracture resistance support reliable everyday use.



Flexible Comfort
Biomimetic elasticity supports an adaptive fit and comfortable wear.



Specifications

Shades : Dark Red, Pink, Light Pink

Net Weight : 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	180-380mPa.s	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	75 (Shore D)	ASTM D2240
Tensile Strength	40-50 MPa	ASTM D638
Elongation at Break	13-17%	ASTM D638
Flexural Modulus	900-1000MPa	ASTM D790
Flexural Strength	70-75 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com

Tray Resin

Tray Resin is a rigid 3D-printing material developed for fabricating patient-specific impression trays. Its low shrinkage and good dimensional stability support accurate tray fabrication, while reliable mechanical strength helps resist deformation during impression procedures. The material produces smooth, durable trays suitable for a range of prosthodontic applications.

Key Performance Benefits



Precision Fit
Low shrinkage and dimensional stability support an accurate fit to the patient's anatomy.



High Rigidity
Helps minimize tray flexing to support accurate impression results.



Specifications

Shade: Blue **Net Weight :** 1000 g / bottle , 500 g / bottle

Technical Data

Physical Properties		
Viscosity	200-350 mPas	
Suitable Wavelength	385nm/ 405nm	
Properties	Value	Test Standard
Hardness	80-90 D (Shore D)	ASTM D2240
Tensile Strength	30-60 MPa	ASTM D638
Elongation at Break	3-15 %	ASTM D638
Flexural Modulus	2200-2700 MPa	ASTM D790
Flexural Strength	>80 MPa	ASTM D790

EVER X-FLOW®
3D Printing Resin

www.atmbio.com